

**BA064**

**EHRlich'S ALDEHYDE REAGENT**

**Formula**

**Reagents:**

HCL, concentrated	100.00 ml
P-dimethyl amino benzaldehyde	4.00 g
Distilled water	100.00 ml

**Directions:**

- 1.Add 1.0 ml of reagent to a 10 ml of urine. Mix well.
- 2.Stand for 10 minutes.
- 3.Observe colour of tube against white background.

**Precautions :**

1. For Laboratory Use.
2. Follow proper, established laboratory procedures in handling and disposing of infectious materials.

**Use:** Detection of urobilinogen in urine.

**Quality Control:**

**Appearance:** Light yellow coloured solution with characteristic odour.

**Results:**

Increased urobilinogen	- Cherry red colour
Decreased or normal urobilinogen	- Absence of colour

**Storage:** Below 30°C.

**Packing :** 100 ml

**Disclaimer:**

User must ensure suitability of the product(s) in their application prior to use. Products conform solely to the information contained in this and other related BIOMARKLABORATORIES publications.

The information contained in this publication is based on our in-house studies and market performance and is to the best of our knowledge true and accurate. BIOMARK LABORATORIES reserves the right to make changes to specifications and information related to the products at any time. Products are not intended for human or animal or therapeutic use but for laboratory, diagnostic, research or further manufacturing use only, unless otherwise specified. Statements contained herein should not be considered as a warranty of any kind, expressed or implied, and no liability is accepted for infringement of any patents.