

TECHNICAL SHEET

BA112	SULPHOSALICYLIC ACID 30%
Formula	
Reagent:	
	Sulphosalicylic acid 30.00 g
	Distilled water 100.00 ml.
Directions:	
	1. Divide urine sample in two portions.
	2. In one portion, add 3 parts of reagent.
	3. Allow to stand for 10 minutes and compare with other portion.
Precautions :	
	1. For Laboratory Use.
	2. Follow proper, established laboratory procedures in handling and disposing of infectious materials.
Use:	To estimate protein content from urine by turbidometric method.
Quality Control:	
Appearance:	Colourless liquid.
Storage:	Below 30°C
Packing:	100 ml bottle

Disclaimer:

User must ensure suitability of the product(s) in their application prior to use. Products conform solely to the information contained in this and other related BIOMARKLABORATORIES publications.

The information contained in this publication is based on our in-house studies and market performance and is to the best of our knowledge true and accurate. BIOMARK LABORATORIES reserves the right to make changes to specifications and information related to the products at any time. Products are not intended for human or animal or therapeutic use but for laboratory, diagnostic, research or further manufacturing use only, unless otherwise specified. Statements contained herein should not be considered as a warranty of any kind, expressed or implied, and no liability is accepted for infringement of any patents.