

**BA112**

**SULPHOSALICYLIC ACID 30%**

**Formula**

**Reagent:**

Sulphosalicylic acid	30.00 g
Distilled water	100.00 ml.

**Directions:**

1. Divide urine sample in two portions.
2. In one portion, add 3 parts of reagent.
3. Allow to stand for 10 minutes and compare with other portion.

**Precautions :**

1. For Laboratory Use.
2. Follow proper, established laboratory procedures in handling and disposing of infectious materials.

**Use:** To estimate protein content from urine by turbidometric method.

**Quality Control:**

**Appearance:** Colourless liquid.

**Storage:** Below 30°C

**Packing:** 100 ml bottle