

BDD023

BILE ESCULIN

SPECIFICATIONS:

APPEARANCE: Sterile 6 mm diameter filter paper disc impregnated with Esculin.

Directions:

Place Bile Esculin disc on the seeded Bile Esculin Agar Base (B424) plate or another media. Incubate at 35-37°C for 18-24 hours.

Cultural response:

Cultural characteristics after 18-24 hours at 35-37°C.

Organism(ATCC)	Esculin Hydrolysis
Streptococcus faecalis (29212)	+
Streptococcus pyrogenes(19615)	-
Listeria monocytogenes (19118)	+

Precautions : For Laboratory Use.

USE: Used for rapid detection of Esculin hydrolysis in presence of bile for differentiation group D Streptococci from non-group D streptococci.

Storage: Between 2 to 8°C.

Packing: 25 discs per vial

Disclaimer:

User must ensure suitability of the product(s) in their application prior to use. Products conform solely to the information contained in this and other related BIOMARK LABORATORIES publications.

The information contained in this publication is based on our in-house studies and market performance and is to the best of our knowledge true and accurate. BIOMARK LABORATORIES reserves the right to make changes to specifications and information related to the products at any time. Products are not intended for human or animal or therapeutic use but for laboratory, diagnostic, research or further manufacturing use only, unless otherwise specified. Statements contained herein should not be considered as a warranty of any kind, expressed or implied, and no liability is accepted for infringement of any patents.