

BF030

BROMOTHYMOL BLUE SUPPLEMENT

FORMULA

Ingredients :

(Per vial, sufficient for 500 ml medium)

Bromo thymol blue 20.00 mg

Directions :

Rehydrate the contents of 1 vial aseptically with 2 ml. of 0.2 N sodium hydroxide. Mix well and aseptically add to 1000ml of sterile, molten CLED Agar Base without Indicator (B959). Mix well and pour into sterile petri plates.

Precautions :

1. For Laboratory Use.
2. Follow proper, established laboratory procedures in handling and disposing of infectious materials.

Use : This supplement is recommended for the isolation and presumptive identification of microflora in urinary tract.

Storage: Between 2 to 8°C.

Packing : 5 vial